








# TOPAZ Control Panel

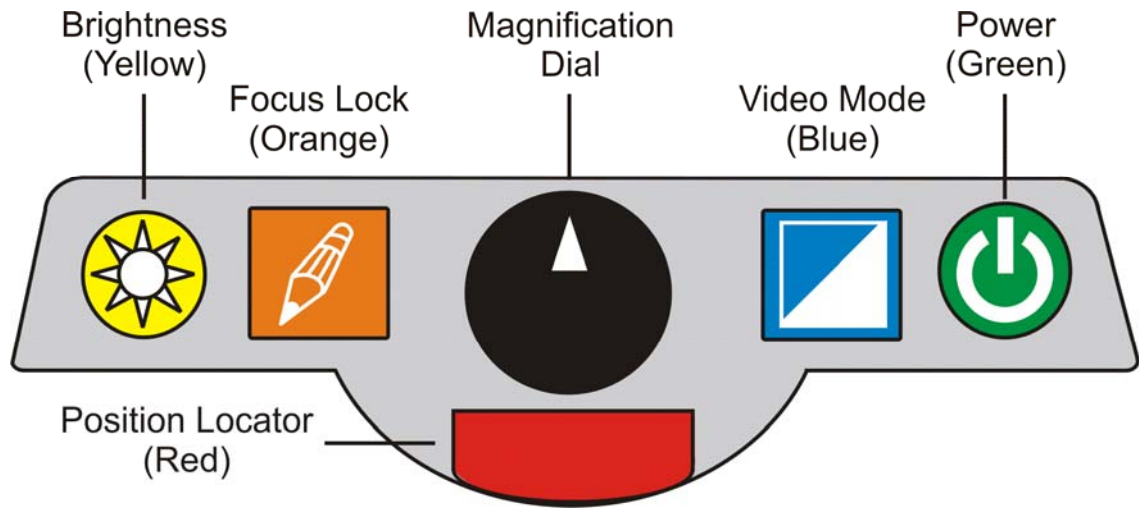

| Dial or Button  | Location   | Description   |
|---|--|---|
|   | The Brightness dial is located at the far left on the control panel.           | Turn this dial to increase or decrease brightness. You can turn the lights on and off by pressing the Brightness dial and the Focus Lock button simultaneously. |
|  | The Focus Lock button is the second button from the left on the control panel. | Press this button to turn Auto Focus on or off. You can turn the lights on and off by pressing the Brightness dial and the Focus Lock button simultaneously.    |
|  | The Magnification dial is located in the center of the control panel.          | Turn this dial to increase or decrease magnification.   |

| Dial or Button  | Location   | Description  |
|---|--|--|
|  | <p>The Position Locator button is located directly underneath the Magnification Dial in the center of the control panel.</p> | <p>Press this button to turn the Position Locator beam on or off.</p>  |
|  | <p>The Video Mode button is the second button from the right on the control panel.</p>                                       | <p>Press this button to cycle through the available video modes.</p>   |
|  | <p>The Power button is located at the far right on the control panel.</p>  | <p>Press this button to turn the power on for the monitor, camera, and lights. Press and hold this button to turn the power off.</p> |